melomind

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Instructions of use of the melomind® solution

Important safety instructions

- Read the instructions
- Keep these instructions
- Pay attention to all warnings
- Follow all instructions
- Do not use this device near water
- Clean only with a dry or damp cloth
- Do not place melomind® near sources of heat
- Use only accessories specified by the manufacturer
- Do not attempt to repair the headset or its accessories yourself. Only qualified personnel may perform maintenance work.

Thank you for choosing melomind®, the first anti-stress solution composed of a headset and a mobile application offering a neurofeedback program dedicated to well-being.

melomind® is the result of research by Cyrebro Neurosignals.

Manufacturer

Cyrebro Neurosignals
28, rue des Petits Hôtels, 75010 Paris, France
SIREN number : 878514686
French company

Scan the QR Code to access the Help Center. Here you will find troubleshooting articles for the mobile application, information on how to use your headset and articles on the science behind melomind®!

Support

For any technical support on the melomind® solution, please contact the Cyrebro Neurosignals customer service at the following email address: support@cyrebro.com

Important notes

melomind® is the first brain training solution dedicated to well-being. melomind® is CE marked.

The declaration of conformity to the requirements is available on our website: <https://www.melomind.care/en/legal-documents/>

TABLE OF CONTENTS

Description	P.30
Before use	P.32
First use	P.33
At each use	P.34
After each use	P.38
Using your melomind® headset as headphones	P.38
Safety and warning	P.40
Compliance	P.45
Legal notice	P.47
After-sale service	P.48

Description

Presentation

Cyrebro Neurosignals is active in research and development in the human and social sciences. It offers the melomind® solution based on the neurofeedback process, which involves training the brain's natural relaxation processes in a non-intrusive way and without side effects.

melomind® is the first anti-stress brain training solution dedicated to well-being. melomind® consists of a Bluetooth headset together with relaxation training based on techniques from the clinical environment. You gradually learn to control your brain activity by strengthening your brain's ability to relax.

From one session to the next, you train your brain's information handling processes. Just like at the gym, after completing a melomind® program, your processes are strengthened and more efficient. Relaxation becomes easy and natural.

By following the melomind® program, you will feel immediate benefits of relaxation and well-being at every session, as well as profound long-term effects on your ability to relax.

The melomind® application guides you day after day through various brain relaxation training exercises. After each session, you discover your phases of deep relaxation, your moments of agitation and rest. You get to know yourself better, to connect to your sensations.

More than an application: melomind® becomes your coach. You receive personalized advice and tailor-made support to carry out your brain training sessions and discover a little more each day how your brain works.

Contents of the box

The solution includes a melomind® headset, electrodes, a cover, a 3.5 mm jack universal audio cable, a USB charging cable with micro-USB connector, a grey protective cover, a user manual (see figure «description of the headset and its accessories»).

1. melomind® headset
2. Electrodes
3. Cover
4. Universal audio cable 3,5mm jack
5. USB charging cable with micro-USB connector

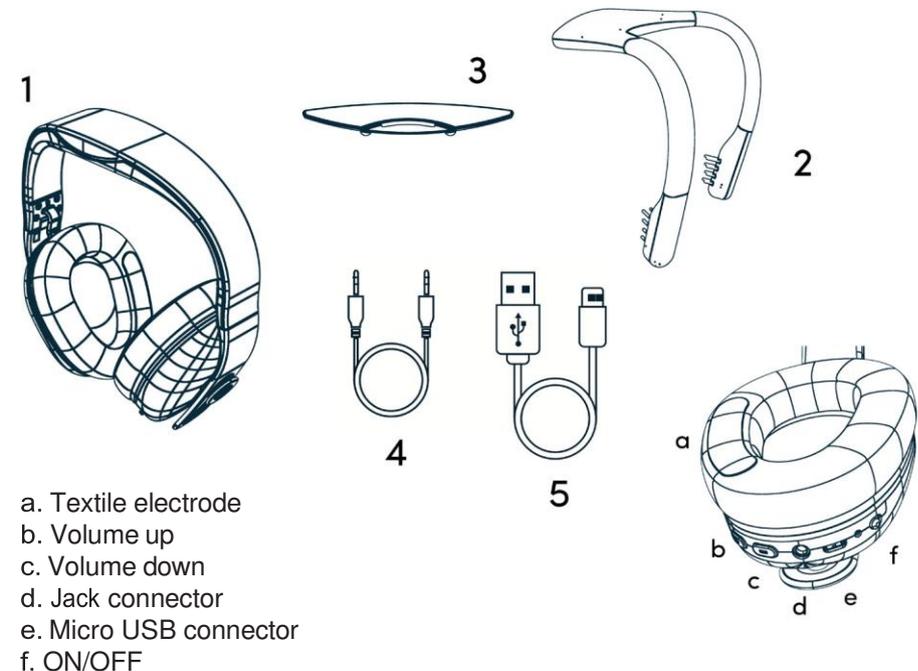
Figure : description of the headset and its accessories

Instructions for use

See [our help center](https://mybrain.atlassian.net/servicedesk/customer/portals) for more information.

Before use

Before starting to use the melomind® solution, it is advisable to fully charge the headset.

Your headset charges in about two hours. It is important to have at least 20% battery power to start a melomind® session. To know the battery level of your headset, connect your headset to the application and go to settings.

Charge your headset

- Insert the micro-USB side of the charging cable into the micro-USB connector located under the left ear cup of the headset.
- Connect the USB-A side of the charging cable to a dedicated USB +5V DC charger.
- The red LED lights up indicating that the melomind® headset is being charged.
- When the red LED goes off, it means that the battery is fully charged.
- Disconnect the charging cable from melomind®.
- The melomind® headset is ready for its first use.

First use

When using the melomind® for the first time, it is essential to follow the next steps to properly set up your headset and mobile device to fully enjoy the melomind® experience.

The melomind® application is available on iOS and Android smartphone and tablet at the following addresses:

- o on the Apple Store if you are an iOS user:
<https://apps.apple.com/fr/app/melomind/id1392413339>
- o on the Play Store if you are an Android user:
<https://play.google.com/store/apps/details?id=mybraintech.com.melomind&hl=en>

Remember to update your application regularly to enjoy the latest features and improvements of the product.

Create and activate your account

- Create and activate your account directly from the application. When you open the application, click on «No account yet? Sign up» at the bottom of the screen. To activate your account, scan the QR code attached to your headset or present on the box of your melomind®. Your headset allows you to activate 5 accounts.
- If you wish to create more than 5 accounts, you can buy additional licenses from our customer service department at support@cyrebro.com. When activating these accounts, do not scan the QR code and click «Activate my license» at the bottom of the screen, and type your license number.

At each use

STEP 1 : Prepare your session


- Make sure your headset is charged.
- Verify that the Bluetooth is turned on on your device.
- Make sure you are connected to the internet.
- For Android users: make sure location is enabled in your device settings. Since the release of version 6.0, Android requires that you have given access to the location of your device to the app and that location is activated in your device settings to connect your headset or any other Bluetooth device. This information is necessary to detect the presence of the headset. This is an Android requirement and the melomind® solution does not process your location data.

STEP 2 : Start your program

Your program is accessible from the application's home page. During this program, you will carry out several sessions during which you will follow different objectives to perform neurofeedback exercises and answer questionnaires to evaluate your progress. This program is divided into several phases. The session to be carried out always appears by default when the application is opened.

STEP 3 : Connect your headset to the application

a) Turn on your headset: turn on your headset by pushing the switch to the «ON» position located under the left ear cup. A flashing blue LED indicates that the headset is on and ready for Bluetooth pairing.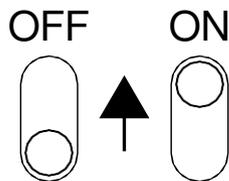

b) Connect your headset: once you are on the «Connect your melomind® headset» screen, click on the QR code pictogram and scan the QR code attached to your headset. Authorization and pairing requests will be sent to you the first time. Please follow the instructions on the screen. The blue LED will stop flashing when the headset is connected to your device via Bluetooth.


Note 1: **Make sure you are connecting from the application and not from your device settings, unless it is required by the application.** In case of problems, consult the help center <https://mybrain.atlassian.net/servicedesk/customer/portals> or contact customer service directly from the application in the help and customer support section.

Note 2: Your melomind® headset has two Bluetooth's. The first allows you to send your brain signal data to the mobile application. In the settings of your phone, it is called melo_XXXXXXXXXX. The second allows the mobile app to send the audio signal to your headset. In your phone settings, it is called MMXXXXXXXXXX.

Note 3: When updating your app or when getting your headset, you may need to update your headset software. It will launch once your headset is connected. Simply follow the instructions in the app. These updates contain improvements and bug fixes.

STEP 4: Adjust your headset

The headset adjustment step provides a good quality signal for your melomind® session. It is important that the headset is well positioned to succeed in this step. Follow the instructions displayed in the application. The following steps should be followed before connecting to the sensors on the application to ensure an optimal signal.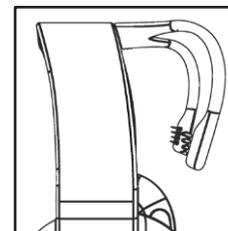


Connect the electrodes to your headset, you should hear a click which means that your electrodes are properly connected. (see figure)

Your melomind® headset has 4 electrodes, the two removable electrodes that are placed at the back of your head and two textile electrodes, present on the headset's pads that are placed behind your ears.

To adjust your headset, these 4 electrodes must be in contact with your skin.

1. Before starting this step, remove your earrings and glasses.
2. Place the headset on your head with your hair as free as possible behind the ear pads.
3. Adjust the height of the headphones so that the headphones are snug. If the ear cushions are not tight enough, contact with your skin will not be possible.
4. Then gently run your fingers under the ear cushions at the back of your ears to make sure there is no hair blocking contact with the textile sensors.
5. Press lightly on the electrodes at the back of your skull so that they make good contact with your scalp.

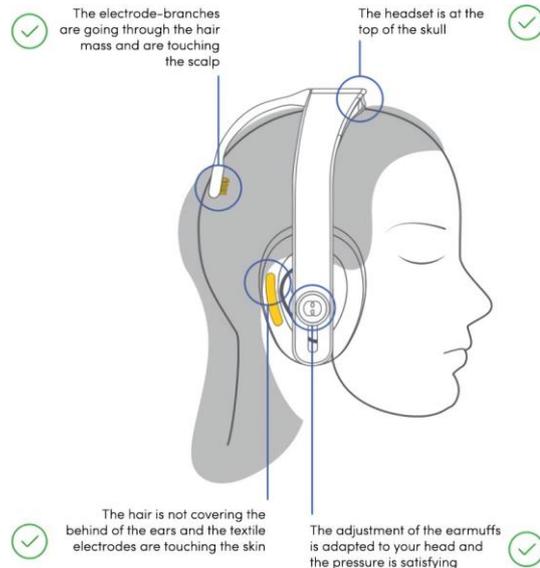6. Make yourself comfortable and stay still. The signal takes a few minutes to stabilize. On your application, the sensors will turn green when the signal is of good quality. This step can take up to a minute and a half.

7. If they do not turn green, reposition the headset again before repeating the adjustment step until you find the perfect position.

STEP 5: Calibrate your headset

During the calibration step, we measure your alpha wave level at rest before you put yourself in a relaxed state for your session. This measurement is fundamental: it will serve as the basis for the personalization of your day's session. The difficulty of the latter will be conditioned by your cerebral state during this step. For an optimal session, you must therefore close your eyes, stay still and do not put yourself in a particular state. Once this step is completed, your session begins and you can finally begin your relaxation.

If you already start the relaxation work during this stage, you increase the difficulty of the session. On the contrary, if you are too agitated, the session will be easier but distorted.

Note 1: If you are unable to perform the calibration, the received signal may not be of good quality. Just as with the adjustment, make sure to position the headset correctly. These two steps are dependent on the signal quality and therefore the positioning of the headset.

STEP 6: Perform your neurofeedback exercise

You will follow a musical session lasting several minutes where you will be immersed in a variety of sound environments, travelling from tropical forests to heavenly beaches. Among the sounds, melomind® will ask you to follow a particular sound. It represents your own level of relaxation. This sound is deep and vibrant like a bass note played continuously.

In order to inform you about the evolution of your relaxation level, this sound will vary in volume. Your brain relaxes: the volume of the indicator goes down. Your brain is more active: the volume of the indicator increases.

STEP 7: Analyze your results

At the end of your exercise, a results page is displayed with the evolution of your state of relaxation during your session.

Remember to ask yourself how you feel and compare the results before and after each session.

After each use

- Turn off your headset.
- Disconnect the pair of electrodes.
- Clean the electrodes with a damp cloth.
- Store the headset and the electrodes in their protective cover.
- Remember to check the battery level before the next use.

Using your melomind® headset as headphones

To use your melomind® headset as headphones, go to your phone or your tablet, select from the Bluetooth settings the reference MMxxxxxxx. You can also use the Jack cable.

REGULATORY ISSUES IMPORTANT INFORMATIONS

Safety and warning

Instructions

Read and understand all instructions in the documentation.

Processing of your personal data

For more information on the data processing implemented by Cyrebro Neurosignals, in compliance with the Data Protection Regulations, please consult our Privacy Policy on our website <https://www.melomind.care>

Battery

Do not replace the melomind® solution headset battery by yourself, as this may damage the battery, which could cause overheating and injury. The lithium-ion polymer battery inside the melomind® headset should only be replaced by Cyrebro Neurosignals or a Cyrebro Neurosignals service provider and should be recycled or disposed of separately from household waste. The battery must not be incinerated. Do not heat the battery or expose the melomind® solution to fire, water or other liquids. Do not put melomind® in the microwave, washing machine or tumble dryer.

Recharge

Do not wear melomind® when charging the battery. When charging the melomind® solution headset, do not connect the mini jack. Charge your melomind® using only the dedicated cable. In order to ensure proper operation and to avoid personal injury and material damage, the USB (the largest connector of the «USB - micro-USB» cable) must only be connected to the USB sockets of approved peripherals such as a desktop computer, laptop, etc. USB outputs from non-approved devices may cause unexpected safety hazards.

Using the wrong cables or adapters or damaged cables or recharging the battery in the presence of moisture can cause electric shock, overheating and injury. Unplug the USB cable once melomind® is fully charged. The micro-USB connector on the melomind® headset must not be interconnected with devices other than the tablet or device where the melomind® solution application is installed, or a computer, etc.

Water, humidity and heat

Do not use melomind® near water or when wet. melomind® is not water-resistant but can be wiped off with a damp cloth for hygiene. The headset and its electrodes must not be exposed to sunlight, fire or any other equivalent heat source.

Radio signals

The melomind® headset solution uses radio signals to connect to wireless networks. Observe signs and notices prohibiting or restricting the use of wireless devices (e.g., health care facilities, airlines, or hazardous locations). Although melomind® has been designed, tested and manufactured in accordance with regulations governing radio frequency emissions, emissions from melomind® may adversely affect the operation of other electronic equipment, causing it to malfunction.

Physical symptoms

In rare cases, people may experience seizures or loss of consciousness due to exposure to flashing lights or patterns created by the display of certain applications on mobile or other devices. If you are prone to this or have experienced nausea, involuntary movements, tingling, numbness, or vision problems while using such devices in the past, you should consult your physician before using similar applications and immediately discontinue use of such applications if symptoms occur. You should avoid prolonged use of such applications to minimize any possible discomfort or fatigue, including any muscle, joint or eye strain, and you should closely monitor your children's use of the technology to avoid potential problems.

Age

The processing of personal data relating to a child is lawful when the person is at least 16 years old (Chapter II - Article 7 - Regulation (EU) 2016/679). However, if the child is under 16 years of age, such processing is lawful only if and insofar as consent is given or authorized by the holder of parental responsibility over the child. If the user is under the age of 18, the user must review all materials and information with his or her parent or guardian to ensure that both the user and the parent or guardian understand it.

Explosive atmospheres

As with all electrical appliances, do not charge or use melomind® in an area with a potentially explosive atmosphere, e.g. in a refueling area or in places where the air contains chemicals or particles (such as grain, dust or metal powders). Observe all signs and instructions.

Choking hazard

The melomind® solution and its accessories may present a choking hazard. Keep these accessories away from young children.

Context of use

Use melomind® in appropriate contexts. melomind® is intended for indoor use in a properly seated position and should not be used while walking or driving. Failure to pay full attention while walking or driving could result in death, serious injury or property damage. You assume total responsibility and risk for not following these instructions.

Repair

DO NOT use tools to disassemble or attempt to repair any part of the device. Do not open, modify or attempt to repair. Do not use melomind® and its accessories if they are damaged. Do not attempt to repair damaged components: only a professional approved by Cyrebro Neurosignals is authorized to perform technical maintenance operations. Any modifications made to the device that are not approved by Cyrebro Neurosignals will void the warranty and the user's authorization to use the device.

Hearing damage

To avoid possible hearing damage, do not listen at high volumes for long periods of time.

Duration of use

The melomind® solution is designed for a daily session of one hour or less. Prolonged daily use for longer than the recommended duration should be avoided.

Operating conditions

The melomind® solution is approved for use under the following conditions:

- Temperature difference:
Storage / Transport: -20 ° C to + 45 ° C.
Use: 0 ° C to + 45 ° C.
- Humidity : relative humidity from 10% to 90% (non-condensing)
- Battery: battery voltage level: 3.6V to 4.2V

For information purposes only

The content found on www.melomind.care websites or found using Cyrebro Neurosignals software products or services is for informational purposes only and is not intended to replace the relationship between you and your physician or other health care provider. Cyrebro Neurosignals does not have a medical license, is not licensed to provide care, and has no expertise in diagnosing, investigating, or treating medical conditions of any kind. **This product is not a medical device and is not intended for medical use, diagnosis or as a therapeutic tool.**

Contraindications

Users must not have a scalp wound or suffer from skin lesions or problems such as blisters, burns, surgical wounds, or other superficial injuries or contagious diseases.

Users must not have any implanted electrical devices such as a pacemaker. The melomind® solution is contraindicated for users who are hypersensitive to electromagnetic fields and other non-ionizing radiation. Adjust the size of your melomind® headset according to the instructions for use. An incorrectly adjusted headset, too tight, can cause headaches.

Compliance

Declarations of conformity

The declaration of conformity and other regulatory documents are available on our website: <https://www.melomind.care/en/legal-documents/> melomind® complies with European requirements (CE).

Radio and electromagnetic interference

Radiofrequency wave emissions from the melomind® solution can interfere with the proper functioning of other electronic equipment and lead to failure. To avoid interference (e.g. with a pacemaker), keep a safety distance of at least 15 cm.


EUROPE

The declaration of compliance with standards can be found on our website: www.melomind.care/pages/legal

Environment

In 2012, the European Union introduced the Waste Electrical and Electronic Equipment Directive (WEEE). The main objective of the directive is to ensure that WEEE is collected and treated separately. WEEE can contain hazardous substances that should not end up in the human environment and can have adverse effects on them if they do.

End users should contact their local government about disposal options if no municipal collection point is available.

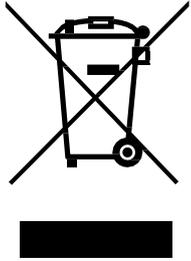This symbol means that this product and the battery it contains must not be disposed of with household waste (2012/19/EU).

Observe the regulations in force in your country for the disposal of electrical and electronic products. Correct disposal helps to protect the environment and your health. The purchase price of the melomind® solution includes an eco-contribution for the treatment, collection and recycling of electronic devices, batteries and packaging.

Legal notice

Copyrights and trademarks

The melomind® brand, Cyrebro Neurosignals®, names and related logos are French trademarks registered by the company Cyrebro Neurosignals.

«Android», «Google», «Google Play» are registered trademarks owned by Google Inc. and registered in the United States and/or other countries. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and are registered in the United States and/or other countries.

Mention of third-party products is for informational purposes only and constitutes neither an endorsement nor a recommendation. Cyrebro Neurosignals assumes no responsibility for the performance or use of these products. The melomind® solution is not intended for any medical use.

Cyrebro Neurosignals disclaims any loss, liability or damage resulting from medical use of the melomind® solution. The terms of use of any Cyrebro Neurosignals product or service are set out in the terms and conditions of use or any other agreement with Cyrebro Neurosignals applicable to the melomind® solution.

Disclaimer

There are no other express or implied warranties other than those described in the general conditions of sale and after-sales service and those of replacement and/or repair described in the special conditions approved by the distributor.

Cyrebro Neurosignals declines all responsibility for any other commitment made on its behalf regarding the product.

Cyrebro Neurosignals declines all responsibility and/or damage resulting from failure to comply with the regulatory notice and instructions for use in the notice, in particular:

- Use that does not strictly follow the previous recommendations, instructions and warnings, use of any accessories not supplied by Cyrebro Neurosignals,
- Damage resulting from disassembling any part of the device,
- Damage resulting from disassembly by any incorrect cleaning method not described in the previous paragraphs.

After-sale service

Warranty and returns

You have legal rights (such as the consumer laws of the country in which you live) («Consumer Rights»). None of your Consumer Rights are limited or excluded.

You must follow Cyrebro Neurosignals return process described in the Terms and Conditions to make a claim. You will find all information related to the warranty and the return of the melomind® solution in the General Terms and Conditions of Sale.

Customer service

If you wish to obtain additional information, for example on how to replace an accessory, or if you encounter a problem, visit our help center at <https://mybrain.atlassian.net/servicedesk/customer/portals> on the website, otherwise contact the customer service directly from the menu of the melomind® application or by email at support@cyrebro.com.

www.melomind.care

support@cyrebro.com